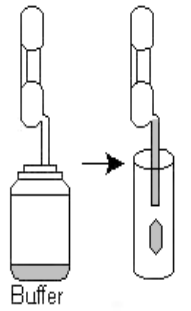
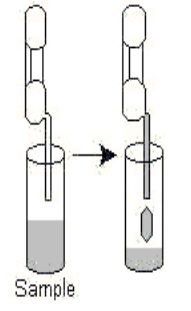
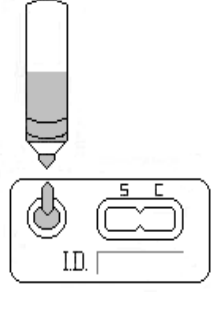
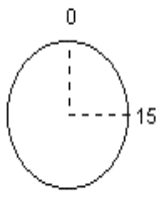
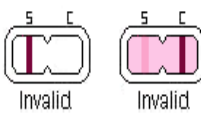
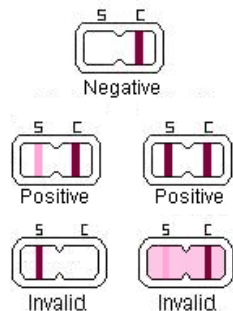


SAS™

FluAlert

CLIA Complexity: Waived
For use with Influenza A or B Test

- Refer to the package insert for complete instructions.
- Bring all specimens and test components to room temperature.
- Use only the pipettes provided in the kit. If bubbles are seen in the pipette stem, throw the pipette away and start with a new pipette.
- The test must pass Procedural QC before reading the results.

<p>STEP 1 Firmly Squeeze the Pipette Bulb and Fill the Pipette Stem with Extraction Buffer. Squeeze and Dispense the Entire Contents of the Pipette into an Extraction Tube</p> 	<p>STEP 2 Using a New Pipette, Firmly Squeeze the Pipette Bulb and Fill the Pipette Stem with Sample Squeeze and Dispense the Entire Contents of the Pipette into the Extraction Tube</p> 	<p>STEP 3 Place a Filter Cap on the Extraction Tube and Dispense 4 Drops of Extraction Sample into Sample Well of the Test</p> 	<p>STEP 4 Wait 15 Minutes</p>  <p>Do Not Read Results After 30 Minutes</p>	<p>STEP 5 Check Procedural QC Positive-A Colored Line Must Appear in the C region of the Test Window. Negative-The Background Must be White to Light Pink and Not Interfere with Reading the Test.</p> 	<p>STEP 6 Read Results</p> 
--	--	--	---	---	--