

# SAS™ Legionella Test



▶ **CPT Code:**  
87449

▶ **Sensitivity:**  
>90.5%

▶ **Specificity:**  
>95%

▶ **Specimen:**  
Urine

SAS™ Legionella Test is a rapid membrane based immunoassay for the qualitative detection of *Legionella pneumophila* serogroup 1 antigens in urine. SAS™ Legionella Test aids in the detection of Legionnaire's Disease.


- CLIA Classification: Moderately complex.
- Simple to run—one-step procedure, results within 15 minutes.
- Disposable specimen droppers included.
- Room temperature storage.



# SAS™ Legionella Test

## Test Procedure

The pouch must be at room temperature before opening to avoid condensation of moisture on the membrane. Allow specimen and/or controls to reach room temperature prior to testing.

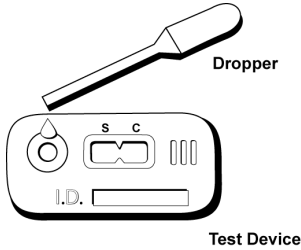


1. Remove the test device from the protective pouch and place it on a flat surface. Label the device with patient or control identifications.
2. Dispense 4 drops (approximately 150µL) of the supernatant into the round sample well (see illustration). Wait for colored lines to appear.

## Test Results




Read results in 15 minutes or less. The presence of the control line is not indicative of the test being completed. Interpretation after 15 minutes may be inaccurate.

The test is negative if a colored line only appears in the C (control) area. The test is positive if one colored line appears in the C (control) area and one line appears in the S (specimen) area.

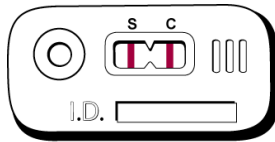

Positive

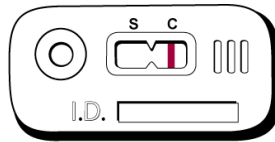

Negative

(See package insert for detailed instructions.)

Product No.

**044020**

20-Test Kit

Product No.

**044413**

Control

(2 × 1 mL,  
2 Vials/Pk)

Description	Cat. No.	Test/ Kit	Specimen	Application/Shelf Life	Sensitivity, Specificity	CLIA/ CPT Code	Results/ Minutes
SAS™ Legionella Test	044020	20	Urine	Detects <i>Legionella pneumophila</i> serogroup 1 antigens (Shelf Life: 18 mos.)	>90.5%, >95%	Moderately complex/ 87499	15
Control Set	044413	2 × 1 mL, 2 Vials/Pk (NOTE: For use with 044020)					

Distributed by:

Manufactured by:



Reliable Diagnostic Tests

4919 Golden Quail, San Antonio, Texas 78240 U.S.A.

(210) 699-8800 • (210) 699-6545 Fax • www.sascientific.com